



# SOLGIENE

ECO-FRIENDLY HEALTH SOLUTIONS

## Solgiene All Purpose Disinfectant Cleaner (concentrate)

An excellent disinfectant cleaner containing G-cide technology

### DESCRIPTION

Solgiene All-Purpose Disinfectant Cleaner is a clear, orange, citrus fragranced liquid mainly used on all hard surfaces. This product contains 2% Glutaral C11-C15 Pareth 9 and all biodegradable ingredients. It is registered as a disinfectant / detergent for general use and can be used to clean up (safely) any blood, body fluid or excreta.

### APPLICATIONS

Solgiene All-Purpose Disinfectant Cleaner is suitable for use in hospitals, theatres, police station mortuaries, clinics, nursing homes, and all high risk areas where instruments and surfaces, which are in contact with blood, are used.

### PROPERTIES

Appearance:	Orange, clear liquid
Solubility:	100% in water and alcohol
pH:	Ideally between 7.5 - 7.8
% active G-cide.	2 - 2.2%

### PRODUCT FEATURES

1. Does not contain any nitrosamine forming ingredients
2. Contains G-cide technology
3. All ingredients biodegrade >90% within 72 hours.
4. Non corrosive
5. Kills all known bacteria, fungi and viruses > 99.9%

### DIRECTIONS FOR USE

1. **HIGH RISK AREAS:**  
Dilute 10 times (100mls of product per 900mls of water).  
Examples of which are theatres, labour wards, recovery rooms, I.C.U., casualty and trauma units etc.
2. **INTERMEDIATE RISK AREAS:**  
Dilute 20 times (50mls of product per 950mls of water).  
Examples of which are orthopedic wards, burn wards, general wards, surgeries, laboratories etc.
3. **LOW RISK AREAS:**  
Dilute 40 times (25mls of product per 975mls of water).  
Examples of which are bathrooms, toilets, kitchens and general house-keeping, receptions, blankets and towels etc.

**General instructions for use:**

Dilute the product as per instructions above and put the solution in a spray bottle.  
Apply to the surface area to be cleaned using a mop/pad/cloth and spread over the surface.  
Allow contact time of five minutes.  
Rinse well with water and allow to dry.

**SAFETY PRECAUTIONS AND FIRST AID**

Ingestion: Do not induce vomiting. Drink large quantities of water.

NOTE TO PHYSICIAN: Probable mucosal damage from oral exposure of large quantity of ingestion of undiluted Solgiene multi-purpose disinfectant cleaner spray may contra-indicate the use of gastric pump.

Eyes: Immediately flush eyes with water and continue washing for at least 2 minutes.

Skin: Wash with plenty of water.

Inhalation: Normally none required, however if needed the person is to be removed to fresh air.

Medical Conditions Generally Aggravated by Overexposure: See above.

**STORAGE**

Store tightly sealed containers in cool, dry area away from direct sunlight and sources of heat. Shelf life of properly stored material is at least one year from date of manufacture

**PACKAGING**

Available pack sizes are 1 L, 5 L and 20 L containers

**Disclaimer**

The information contained herein is based on the present state of information and knowledge available. The information provided are guidelines and recommendations and **does not** constitute any guarantee or warranty. Noble Techniques Ltd does not accept any liability of any kind for any loss including, without limitations, consequential loss, injury or damage arising from the use of its products and or information contained herein.

REF: SOL401  
DATE: 08.02.2020